IMMUNO CONCEPTS Serum Diluent

For in vitro diagnostic use **IVD**

OPTIONAL COMPONENTS

Cat. No 100 200 500 1000

INTENDED USE: This component is for use in the Immuno Concepts Test Systems.

This component has been tested in the Immuno Concepts Test Systems and found to meet performance requirements; it has not been tested for performance in other manufacturer's test systems.

Use: Use the Serum Diluent reagent only as a diluent for patient samples. Do not substitute this reagent for PBS in the wash steps of the test procedure. Use of this reagent in the wash step may result in less than optimum results.

Storage: All components can be stored under refrigeration at 2-10°C.

Stability: All components remain stable at least 12 months from date of manufacture. Do not use any component beyond its expiration date.

Instructions for use:

1. Make a 1:10 dilution of the Diluent Concentrate using reconstituted Immuno Concepts PBS.
2. The Serum Diluent is stable for four weeks refrigerated at 2-10°C.
3. Dilute patient samples as directed in the package insert provided with your test kit using the Serum Diluent in place of the Immuno Concepts PBS.

PRECAUTIONS

Sodium azide (0.09%) is used as a preservative. When disposing of reagents, flush with ample volumes of tap water to prevent potential residues in plumbing. Sodium azide is a poison and may be toxic if ingested.

FOR TECHNICAL ASSISTANCE: 1-800-251-5115 (Toll Free) or e-mail: technicalsupport@immunoconcepts.com

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MDSS GmbH