IMMUNO CONCEPTS Optional Liquid Components

For in vitro diagnostic use **IVD**

For Professional Use

INTENDED USE: This component is for use in the Immuno Concepts Test Systems.

This component has been tested in the Immuno Concepts Test Systems and found to meet performance requirements; it has not been tested for performance in other manufacturer's test systems.

Use: All components come ready to use with no aliquoting or reconstitution required.

Storage: All components can be stored under refrigeration at 2-10°C.

Stability: All components remain stable at least 12 months from date of manufacture. Do not use any component beyond its expiration date.

Counterstain: Catalog No. 1105 (4).

Reactive Component: Counterstain, Catalog No. 1105

Ready-to-use dropper vial containing 5.0 ml Evans blue solution (0.5% in PBS).

DANGER: Carcinogen. This reagent should be handled with care. Keep out of the reach of children. If exposed or concerned, get medical attention.

Optional Counterstain: Add 5-10 drops counterstain (0.5% Evans blue) per 100 ml PBS prior to slide immersion. Because the degree of counterstaining desired may vary among individuals, counterstaining intensity may be increased or decreased by simply adjusting the number of drops added to the PBS in this wash.

Semi-Permanent Mounting Medium: Catalog No. 1111 (4).

Non-Reactive Component: Semi-Permanent Mounting Medium, Catalog No. 1111

Ready-to-use dropper vial containing 5.0 ml glycerol-based mounting medium, pH 9.1 ± 0.2. This reagent contains 0.09% sodium azide as a preservative.

PRECAUTIONS

1. Sodium azide (0.09%) is used as a preservative. When disposing of reagents, flush with ample volumes of tap water to prevent potential residues in plumbing. Sodium azide is a poison and may be toxic if ingested.

FOR TECHNICAL ASSISTANCE: 1-800-251-5115 (Toll Free) or e-mail: technicalsupport@immunoconcepts.com

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MDSS GmbH