IMMUNO CONCEPTS nDNA Optional Components

For in vitro diagnostic use **IVD**

Component, Composant, Componente, Komponente:

Cat#, REF

Control Sera / Sérums de Contrôle / Sieri di Controllo
Sueros de Control / Kontrollseren / Kontrollsera: **3021, 3026, 3031**

Fluorescent antibody reagents / Réactif Immunofluorescent
Coniugato FITC / Anticuerpo Conjugado con FITC
Fluoreszenz-Antikörperreagenz / Fluorescerande Antikörperreagens: **3009, 3009G, 3075, 3075G**

Enzyme antibody reagents / Réactif Immuno-Enzymatique
Reagente Anticorpo Enzimatico / Anticuerpo Conjugado con Enzima
Enzym-Antikörperreagenz / Enzymantantikörperreagens: **5009, 5075**

nDNA slide / Lame de ADNn / Vetrino con nDNA / Portaobjeto de nDNA
Objektträger-nDNA / Objektglas-nDNA **3007, 3014**

INTENDED USE: This component is for use in the Immuno Concepts nDNA Test Systems.

This component has been tested in the Immuno Concepts nDNA Test Systems and found to meet performance requirements; it has not been tested for performance in other manufacturer's test systems.

Use: All components come ready to use with no aliquoting or reconstitution required (except the Titratable Control Serum which is treated as an undiluted patient sample.).

Storage: All components can be stored under refrigeration at 2-10°C.

Stability: All components remain stable at least 12 months from date of manufacture. Do not use any component beyond its expiration date.

PRECAUTIONS

1. All human source materials used for this product have been tested and found to be negative (not repeatedly reactive) for antibodies to Human Immunodeficiency Virus-1 (HIV-1), Human Immunodeficiency Virus-2 (HIV-2), hepatitis C virus (HCV) and for hepatitis B surface antigen (HBsAg) by FDA approved methods. However, no test method can offer complete assurance that HIV-1, HIV-2, hepatitis C, hepatitis B, or other infectious agents are absent. Therefore, all kit materials should be handled in the same manner as potentially infectious material.

2. Sodium azide (0.09%) is used as a preservative. When disposing of reagents, flush with ample volumes of tap water to prevent potential residues in plumbing. Sodium azide is a poison and may be toxic if ingested.

FOR TECHNICAL ASSISTANCE: 1-800-251-5115 (Toll Free) or e-mail: technicalsupport@immunoconcepts.com

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MDSS GmbH