

# IMMUNO CONCEPTS Wash Buffer Concentrate

For in vitro diagnostic use **IVD**

## OPTIONAL COMPONENTS

**Cat. No** 1031

**INTENDED USE:** This component is for use in the Immuno Concepts Test Systems.

This component has been tested in the Immuno Concepts Test Systems and found to meet performance requirements; it has not been tested for performance in other manufacturer's test systems.

Use: Wash Buffer Concentrate is used to make Wash Buffer to wash strips.

Storage: Store the Wash Buffer Concentrate under refrigeration at 2-10°C.

Stability: Wash Buffer Concentrate remains stable at least 12 months from date of manufacture. Do not use any component beyond its expiration date.

### Instructions for use:

1. Add the entire contents of one bottle of Wash Buffer Concentrate to one liter of dissolved PBS. Mix well
2. The Wash Buffer Solution may be covered and stored at 2-25°C up to four weeks or until signs of contamination or other visible changes occur. Wash Buffer Solution must be at room temperature (18-25°C) before use.
3. Wash strips as directed in the package insert provided with your test kit.

**FOR TECHNICAL ASSISTANCE:** 1-800-251-5115 (Toll Free) or e-mail: [technicalsupport@immunoconcepts.com](mailto:technicalsupport@immunoconcepts.com)

In the event of damage to the protective packaging, please contact Immuno Concepts prior to use.



Manufacturer



In Vitro Diagnostic Medical Device



Contains Sufficient for <n> tests



Temperature Limitation



Consult Instructions for Use



9825 Goethe Road, Ste. 350  
Sacramento, CA 95827